

# More Than Mobility Inc

www.morethanmobility.ca

For further information or to request a Technical Manual within the United Kingdom please contact: -

Ortho Europe Ltd.  
Mill Lane  
Alton  
Hants,  
GU34 2PX

+44 (0) 1420 83294  
+44 (0) 1420 80068

E-Mail: [sales@ortho-europe.co.uk](mailto:sales@ortho-europe.co.uk)  
Web: [www.ortho-europe.co.uk](http://www.ortho-europe.co.uk)

Outside the United Kingdom please contact your local Ortho Europe office or distributor.

#### Warranty

Ortho Europe components are covered by 24 months limited warranty where the adjoining component is part of the Eco or Premier range, otherwise a 12 month limited warranty applies.  
The warranty requires fitting and use as set out in the above instructions.  
The warranty is void if fitting and routine care instructions are not followed.

Manufactured by : - Ortho Europe Ltd  
Ortho House,  
Nuffield Way  
OX14 1RL.  
United Kingdom



4LA201 Iss.3.01

07/05

Ortho Europe Ltd


ECO system

Instructions for use

Endoskeletal Ankle

4LA201

For Patients up to 100kg

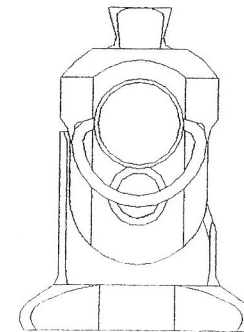

**ORTHO**EUROPE

## ASSEMBLY AND FITTING

### Ankle Adjustment

Changing the side of the 'D' locking ring

To convert from right to left side operation, unscrew and remove the large headed hex screw on the opposite side to the D ring. Withdraw the latching spindle by pulling the D ring. Re-insert the spindle from opposite side of the ankle taking care to ensure that the stop pin in the spindle end face engages the limit groove in the ankle body. Apply a suitable thread-locking compound (eg Loctite 243) to the hex screw threads, and relocate.

### Attachment to foot

The foot is attached to the ankle using the anti-rotation dowel pins supplied and the stainless steel M10 bolt provided.

The foot bolt must be secured with a suitable bonding agent (Loctite 243). If the pivot pin is removed ensure the M5 securing set screws are only finger tight when refitting. Secure with suitable bonding agent (Loctite 243).

Because of the highly aggressive properties of chlorinated and sea water, the set screws for the pyramid attachments must be of stainless steel.


### Aftercare

Ensure the user is aware of the following information:-


Before use check the ankle for signs of corrosion. Refer to the prescriber if the ankle does not function correctly or shows signs of significant corrosion. If the ankle is used in chlorinated water, brine or sea water it must be cleaned thoroughly with tap water after use.

If the ankle position is changed you must make sure that the pin turned

### Technical Data

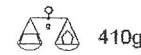


410g

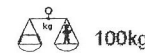


100kg




Low

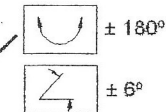

± 180°

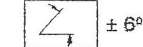

± 6°

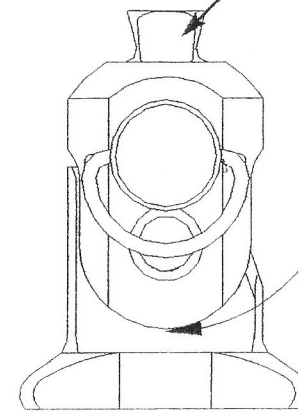

Rotate handle to  
release ankle joint



THREADLOCK  
243



20 Nm



8 mm



For water leisure activity only  
not for daily use

'Compatible with product from the ECO range'

For further information please refer to the Technical Manual

# More Than Mobility Inc

www.morethanmobility.ca